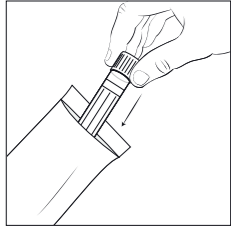
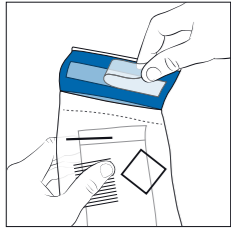
F. Inpakken en versturen / Wrap and send

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
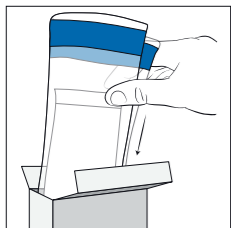
with the tube into the special plastic bag (Safetybag). Close the bag using the adhesive strip.

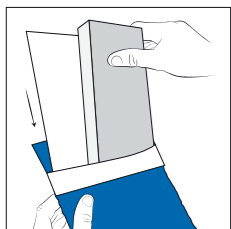
12. Stop de goed afgesloten Safetybag en sluit de doos.
Place the well-sealed Safety bag in the box and close the box.

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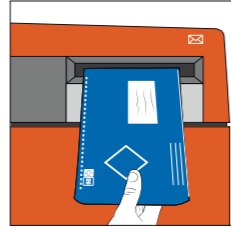your doctor's request form into the return envelope. Close the envelope using the adhesive strip.

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Important: You should post the return envelope the same day, before the last collection. You do not need a stamp.*

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* If you are unable to send the envelope within 24 hours, store the material in the fridge until you are able to send it.

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Questions?

If you have any questions, please contact your doctor/GP or our Customer Contact Centre. The RHMDC Customer Contact Centre is available from Monday to Friday, from 8 a.m. to 5 p.m., tel. 088 260 4000.

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STD test - swab test (eSwab)

This test serves to diagnose sexually transmitted diseases (STDs). To do the test, you must take a swab from your vagina or anus. It does not hurt and will take about 5 minutes. You can do the test at any time of the day, including when you are having your period.

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A. Package content

- Instruction leaflet
- Barcode sheet
- Request form
- Tube and cotton swab (eSwab)
- White bag
- Plastic bag
- Return envelope

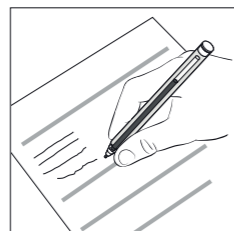
B. Important

Read the full explanation in full on both sides of this instruction leaflet.

Carry out steps 1 to 14.

Follow the instructions that are relevant to you (see D1 and D2). If you need to take a swab of both the vagina and the anus, you must use a separate cotton swab for each body area.

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
Check whether your data have been entered on the doctor's request form and if necessary complete them. As soon as you take the swab, you should note the date and time on the request form.

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
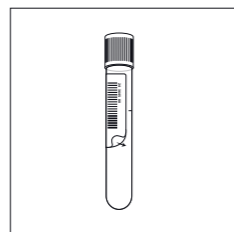
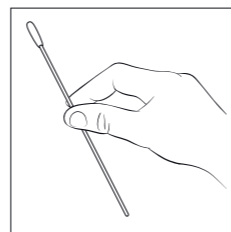
Pick up the cotton swab. Don't touch the cotton! As soon as you drop or put down the cotton swab, you should dispose of it.

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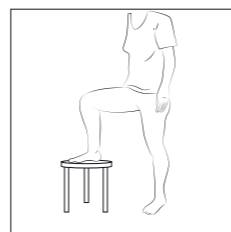
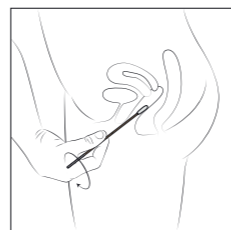
Insert the cotton swab about 5 to 6 cm into the vagina. Twist the cotton swab carefully 5 times.

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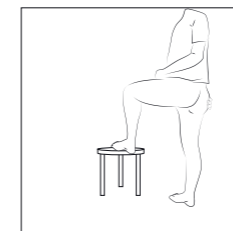
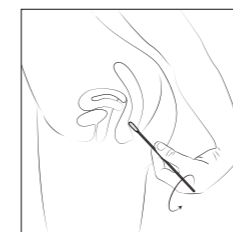
Insert the cotton swab about 3 to 5 cm into the anus (as you would with a thermometer), and twist the cotton swab carefully 5 times.

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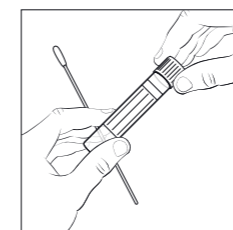
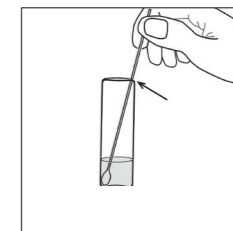
Hold on to the cotton swab in the middle - do not touch the cotton. Pick up the tube and untwist the lid. Leave the liquid in the tube.

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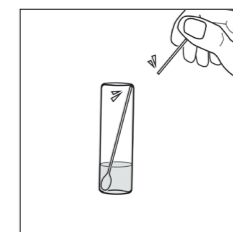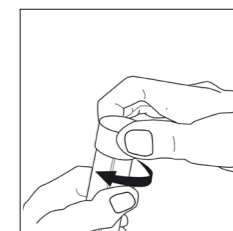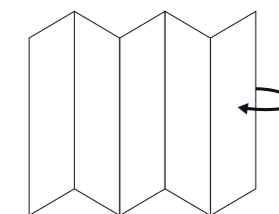
The explanations are on the front and back of this instruction leaflet.